LAICA

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HC57/c - 08/2016

LAICA

LAICA S.p.A.
Viale del Lavoro, 10 - Fraz. Ponte
36021 Barbarano Vicentino (VI) - Italy
Tel. +39 0444.795314 - 795321
Fax. +39 0444.795324
Made in China

www.laica.com

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This device, completely automatic, is used to measure and control in a non-invasive way the value of blood pressure (systolic and diastolic), the heart rate and the presence of arrhythmias. Blood pressure is the force exerted by blood on the arteries.

This force varies constantly depending on heartbeat. When the heart contracts, blood pressure is at its highest (systolic blood pressure value) whilst at the end of the heart's "relaxation period", the value is at its lowest (diastolic blood pressure value).

Blood pressure is influenced by many factors: smoke, stress, physical activity, eating, the time of day...

The arrhythmia, or irregular heart rate, can be caused by frequent anxiety statuses, specific emotional statuses, excessive use of alcohol, thyroid noises, etc. It can be a symptom of a specific physical or psychological condition (temporary noise) or of a real and proper heart disease.

Always refer to a physician in case the blood pressure monitor shows the symbol for irregular heart rate, in order to establish the most suitable treatment to be followed.

Accurate and easy to use, this device enables even the elderly and less expert to measure blood pressure in the comfort of their own home; its quick and reliable results enabling pressure values to be checked daily.

Laica S.p.A. - Viale del Lavoro, 10 - Fraz. Ponte
36021 Barbarano Vicentino (VI) - ITALY
Made in China

BEFORE STARTING TO USE THE DEVICE, PLEASE READ THIS INSTRUCTION MANUAL CAREFULLY FROM COVER TO COVER AND KEEP IT SAFE FOR FUTURE REFERENCE.

Warning:

- Self-measuring means checking, and is no substitute for diagnosis or treatment. Unusual values should always be discussed with your doctor.
- Under no circumstances should doses of any medicine prescribed by your doctor be altered.
- The heartbeat display is not suitable for checking the frequency of pacemakers!
- Should there be a deviation from the normal rhythm of the heart (arrhythmia) or other cardiac problems, for carriers of pacemaker, for those who use anticoagulants, for women in state of pregnancy, the measurements made with this device should be evaluated only after consulting

your doctor.

In such cases compare the results with the doctor in order to verify their correctness.

- In case of heavy arteriosclerosis and other circulation problems or spasms in the upper limbs the readings could result incorrect.
- This device cannot be used within the area interested by radiations of Computerized Axial Tomography (TAC), X rays and Magnetic Resonance (RMN).
- This appliance must be used solely for the purpose for which it was designed and in accordance with the operating instructions.

Any other use shall be considered improper and hence hazardous.

The manufacturer cannot be held responsible for any damage resulting from improper or incorrect use.

WARNING

- 1) If there is reduced blood circulation in the arm due to acute or chronic vascular diseases (including the restriction of the diameter of the blood vessels), the precision of the measurement could be affected.
- 2) In the case of cardiocirculatory illnesses, the measurements could be falsified; the same could happen with very low blood pressure, problems with blood supply, arrhythmia or other pre-pathological conditions.
- 3) Do not use near to strong magnetic fields. It must be kept well away from radio equipment or mobile phones.
- 4) Pressure varies from person to person, and increases or decreases every day, depending on the stress the heart is subjected to.

Normally pressure is low during the night, during sleep or during relaxation, and it is higher during the day, when we smoke, during physical exercise, under effort, when we are anxious or tense. Blood pressure tends to increase with age and depends greatly on the individual way of life. It is well known that stress, obesity, excessive intake of salt and alcohol all contribute to increase arterial pressure.

5) Keep the batteries out of the reach of children.

CLASSIFICATION OF BLOOD PRESSURE VALUES

The following table has been developed by the World Health Organization (WHO), a special agency

of the United Nation for health issues.

RANGE	SYSTOLIC	DIASTOLIC	MEASURES
Optimal	105 - 120	60 - 80	self-measurement
Normal	120 - 129	80 - 84	self-measurement
Limit range	130 - 139	85 - 89	refer to a physician
High blood pressure (1st level)	140 - 159	90 - 99	refer to a physician
High blood pressure (2nd level)	160 - 179	100 - 109	refer to a physician
High blood pressure (3rd level)	beyond 180	beyond 110	refer to a physician

At the end of each measurement, the measured pressure data are compared with the above mentioned table. The segments appearing on the left of the display indicate the classification of the blood pressure values:

Optimal	green color
Normal	green color
Limit range	green color
High blood pressure (1st level)	yellow color
High blood pressure (2nd level)	orange color
High blood pressure (3rd level)	red color

Measured values lower than 105 mmHg (systolic) and than 60 mmHg (diastolic) indicate a low blood pressure status. It is suggested to refer to a physician.

CORRECT MEASURING METHOD

Many factors determine variations in arterial pressure: physical exercise, eating, agitation, changes in temperature and emotional stress.

Daily fluctuations of 25 / 50 mmHg are common.

In order to get an accurate blood pressure reading, follow the instructions below:

- 1) Sit on a chair, relax and try to keep still for at least 5/10 minutes before taking a reading.
- 2) Remove clothing and jewellery from the arm or wrist before applying the armband.
- 3) Avoid eating, smoking and drinking (especially alcohol) before taking the measurement. All these factors will affect the result of the measurement.
- 4) Rest the arm on the table so that the armband is on a level with the heart. The arm must be

stretched out naturally and must be kept still.

- 5) Use the case or a similar object to support the forearm.
- 6) Place both feet on the ground.
- 7) **Remember that the blood pressure changes during the day. Try to effect more measurements during the day for better understanding the reactions of your body to the different moments of your day (after a physical activity, during the job, after the meals...).**
- 8) **Do not take only one measurement. At least two measurements should be taken, wait at least 10/15 minutes between the two measurements. More measurements, taken over a long period of time will give you a better indication of your blood pressure.**
- 9) Should an unpleasant feeling occur during the measurement, stop immediately the device with the "O/I" button.

IMPORTANT NOTE!

- The device must be kept at the same height of the heart at all times; otherwise it is not possible to get accurate measurements.
- This unit is designed to measure adult pressure values to be used for people of 18 years and over. If this instrument is applied to the arm of a child, the value measured may be inaccurate. Consult your doctor for measuring the arterial pressure of a child.
- We recommend using the left arm for taking the measurements.
- If the appliance is used in temperature and humidity conditions outside the limits given in the "Technical Features" the readings may not be accurate.

INSERTING/REPLACING BATTERIES

- 1) Open the battery compartment by lifting the cover.
- 2) When installing or replacing the two batteries, insert them in accordance with the polarity marked.
- 3) Place the cover back to its position. At this point, proceed with the setting of date and time, as shown in the paragraph "SETTING MEMORY ZONE AND DATE/TIME". When the battery symbol appears on the display, the batteries must be replaced as they are

flat. Remove the batteries whenever the instrument is not going to be used for a long time.

- 4) Remove the run down batteries by lifting the cover of the battery compartment. Extract the batteries and dispose them as special waste at the specific collection points forecast for recycling. For more information about the disposal of run down batteries, please contact either the store where the device containing the batteries was purchased, or the Municipality, or the waste disposal local service. **Caution: For greater safety, children younger than 12 years should not remove the batteries.**

SETTING MEMORY ZONE AND DATE/TIME

- 1) With the unit off, press the "O/I" button for 3 seconds, the display will flash the last memory zone selected (1 or 2).
To select the memory zone desired, press the "M" key and confirm your choice by pressing the "O/I" button.
At this point, the display flashes the number for the year (**fig.1**). Use the "M" key to adjust the figure, holding it down will speed up the scrolling of the numbers. Press the "O/I" button to confirm it.
- 2) Now the month flashes: adjust with the "M" key and confirm by pressing the "O/I" button.
- 3) Now the day flashes: adjust with the "M" key and confirm by pressing the "O/I" button.
- 4) Now the hour flashes: adjust with the "M" key and confirm by pressing the "O/I" button (the display shows the time in 12 and 24 hour formats, see step 6 to select the desired format).
- 5) Now the minute flashes: adjust with the "M" key and confirm by pressing the "O/I" button.
- 6) At this point, the display shows "US" or "EU" that indicates the format of date and time (US = 12-hour format, EU = 24-hour format). Select the format with the "M" key and confirm by pressing the "O/I" button.
- 7) Press the "O/I" button for 3 seconds to confirm the data and turn off the display.

When you insert or replace the batteries you need to reset the memory zone, date and time as described in steps 1 to 7.

APPLYING THE WRIST CUFF

These instructions must be followed in order to take measurements on either your right or left wrist. **We recommend measuring on the left wrist, but always use the same one even if the right is used.**

- 1) With the palm of your hand facing up, apply the instrument so that the edge of the cuff is approx. 1-2 cm from your palm.
- 2) Apply the wrist cuff around your bare wrist and fasten it in place with the Velcro fastening.
- 3) The wrist cuff must be snug around your wrist, but not too tight.

TAKING A MEASUREMENT

- 1) After the unit has been placed on the right wrist or the left, hold down the "O/I" button until you hear a beep.
On the display, for a few seconds, all the function symbols come on. The up arrow symbol (▲) indicates that the unit is ready for measurement.
At this point the cuff will start to inflate automatically and will stop inflating once the optimal level is reached.
Try to stay relaxed, without speaking or moving.
Caution: When there is still air in the cuff from the previous measurement, the down arrow symbol (▼) will flash on the display until the unit stabilises the pressure.
- 2) The display will show the following values: systolic, diastolic, pulse rate, date and time of the measurement (♥). The heart symbol will appear only in the event that an irregular heartbeat has been detected (≥(♥)≤).
Segments that appear on the left of the display indicate the classification of blood pressure values (see the "CLASSIFICATION OF BLOOD PRESSURE VALUES" section). Once measurement is completed, the unit will beep three times.

Automatic re-pressurisation

The starting pre-set pressure level is 190 mmHg.
If the pre-set pressure is insufficient in the preliminary reading phase or if the hand or wrist is moved, the unit will start inflating again until 220 mmHg are reached. Automatic re-pressurisation is repeated until a valid reading is taken.
However this is not sign of a fault with the appliance.

- 3) If you want to stop measuring for any reason, simply press the "O/I" key and remove the cuff. The unit switches off automatically if it is left idle for 1 minute, otherwise you can switch it off by pressing the "O/I" key.

Make sure the batteries are charged.
Low or dead batteries decrease the efficiency of the pump, which then cannot provide enough pressure in the preset time interval.
For this reason, ERR will appear. Replace the batteries.

MEMORY FUNCTION

This unit has the capacity to store up to 60 measurements in each memory zone (2 memory zones). After each measurement, the following values will be stored automatically: diastolic, systolic, pulse rate, date and time, memory number, segments for classifying blood pressure values (see "CLASSIFICATION OF BLOOD PRESSURE VALUES" section) and the possible presence of irregular heartbeat (arrhythmias) (≡(♥)≡) when the heart symbol is shown.

To recall the stored measurements in the last memory zone selected, you must press the "M" key. On the display will appear (1) or (2).

To scroll through the data in memory press the "M" key again (each press of the key scrolls through the data) and there will appear in succession:

- the word "AVG" which indicates the average of the last 3 measurements
- in the lower right of the display the memory number
- the stored values

The number "60" is the oldest measurement, while the number "1" indicates the most recent value. Switch off the appliance by pressing the "O/I" button.

After more than 60 measurements, the oldest data are automatically cancelled. .

To recall the memory area other than the last one selected before the last measurement, with the unit off it is necessary to hold the "O/I" button for 3 seconds.

On the display the last memory zone selected will flash (1) or (2). Press the "M" key to select one of the two memory zones and confirm the selection by pressing the "O/I" button. At this point, turn off the unit by pressing and holding the "O/I" button for about 3 seconds.

DELETION OF STORED DATA

Deleting all data stored in a memory zone.

When the appliance is turned off recall your memory zone by pressing the "O/I" button for 3 seconds and then the "M" key to select the memory zone and confirm by pressing the "O/I" button. Then switch off the unit, holding down the "O/I" button for about 3 seconds.

At this point, press the "M" key and then the "O/I" button for about 3 seconds, the unit will emit a short beep and the display will show "---". Now all the data in the previously selected memory zone will have been deleted.

INDICATORS

The following symbols may appear on the display:

- : MEMORY AREA No.1: It appears when the measurement is stored in the first memory area.
- : MEMORY AREA No.2: It appears when the measurement is stored in the second memory area.
- Err : no measurement could be taken as the arm moved, incorrect position, other ...see section entitled "CORRECT MEASURING METHOD".
- : batteries finished. Replace batteries.
- : pulse frequency symbol.
- ≡(♥)≡ : indicator of irregular pulse
- : reference segments for the classification of the blood pressure values
- "AVG" : indicates the average of the last 3 measurements

CARE AND MAINTENANCE

- This unit contains precision components.
Hence, care must be exercised to avoid exposing the unit to extreme changes in temperatures, humidity, knocks, dust and direct sunlight. Do not drop or hit the unit.
- Clean the unit using a soft, dry cloth only.
Do not use any chemical or abrasive products to clean the monitor.

Ensure that no liquids enter the appliance.

- **This appliance is not waterproof.**
- Place the device in the supplied case and, to store it, observe the environmental conditions stated here below.

SPECIAL CASES

If the following circumstances occur:

- 1) After pressing the "O/I" key the reading does not start.
 - Check the batteries and replace them if needed.
 - Clean the contacts of the batteries with a dry cloth.
- 2) Nothing appears on the display when the "M" key is pressed.
 - The batteries are flat.
- 3) The batteries do not last long.
 - use alkaline batteries to increase working autonomy. If normal zinc-carbon batteries are used they will need changing more often.
- 4) You cannot complete a reading.
 - check the batteries and change them if needed.
- 5) The blood pressure levels are different with each reading - the readings are very low or very high.
 - the measuring position is not correct - Read the paragraph "Correct measuring position" again.
 - the blood pressure readings vary constantly depending on the time of day and the state of nerves of the person. Read the paragraphs "Warnings" and "Important" again.
- 6) The heart beat readings are too high or too low:
 - The person moved while the reading was being taken.
 - The reading was taken after physical exertion.
 Read the paragraphs "CORRECT MEASURING METHOD", "WARNINGS" and "IMPORTANT" again.
- 7) On the display, appears the symbol $\approx(\heartsuit)\leq$.
 - It has been detected the presence of an irregular pulse (arrhythmias). Repeat the measurement. **If the symbol $\approx(\heartsuit)\leq$ appears again, refer to a physician to establish the most suitable treatment to be followed.**

COMPONENTS

- 1) LCD DISPLAY
- 2) "O/I" KEY
- 3) M KEY
- 4) ARMBAND
- 5) BATTERY COMPARTMENT
- 6) BATTERIES
- 7) CASE

TECHNICAL FEATURES

- Pressure indication range: between 0 and 300 mmHg (arm cuff pressure)
between 30 and 180 pulsations/min. (pulse frequency)
- Precision: ± 3 mmHg (arm cuff pressure)
 $\pm 5\%$ of the read value (pulse frequency)
- Inflating and deflating: automatic
- Power supply: 2 alkaline batteries type AAA (LR03)
- Memories: 60 Memories per area (2 areas)
- Working conditions: between $+5^{\circ}\text{C}$ and $+40^{\circ}\text{C}$; 85% relative humidity or lower
- Storage conditions: between -10°C and $+50^{\circ}\text{C}$; 95% relative humidity or lower
- Wrist circumference: between 135 and 215 mm approx.
- Dimensions: 77x62x33 mm
- Weight: about 120 g (without alkaline batteries)

Attention. Carefully check the instructions for use.

Type BF appliance

IP22: Degree of protection of electrical equipment covers, where the first figure indicates the degree of protection against penetration by solid foreign bodies (from 0 to 6) and the second figure the degree of protection against penetration by liquids (from 0 to 8).

ELECTROMAGNETIC COMPATIBILITY

This device complies with current regulations relating to electromagnetic compatibility (EMC)

and has been designed to be used in the home.

Its emissions are extremely small and are unlikely to generate interference with other equipment. In the event that it is to be used in close proximity to other equipment, it is recommended to follow the instructions given in the tables at the end of the instructions for use.

Certain mobile communications devices and radio systems may have an effect on the operation of this device.

In cases where there are anomalies during operation, it is recommended to distance the device from any other equipment that could cause electromagnetic interference and check for the restoration of performance.

In all cases, in the event of doubt, contact Customer Support or see the tables relating to electromagnetic emissions at the end of these instructions.

DISPOSAL PROCEDURE (Dir. 2012/19/Eu-WEEE)

The symbol on the bottom of the device indicates the separated collection of electric and electronic equipment.

At the end of life of the device, do not dispose it as mixed solid municipal waste, but dispose it referring to a specific collection centre located in your area or returning it to the distributor, when buying a new device of the same type to be used with the same

functions.

If the appliance to be disposed of is less than 25 cm, it can be returned to a retail location that is over 400 m² without having to purchase a new, similar device.

This procedure of separated collection of electric and electronic devices is carried out forecasting a European environmental policy aiming at safeguarding, protecting and improving environment quality, as well as avoiding potential effects on human health due to the presence of hazardous substances in such equipment or to an improper use of the same or of parts of the same.

Caution! The wrong disposal of electric and electronic equipment may involve sanctions.

WARRANTY

This equipment is covered by a 2-year warranty starting from the purchase date, which has to be proven by means **of the stamp or signature of the dealer and by the fiscal payment receipt that shall be kept enclosed to this document.** This warranty period complies with the laws in force and applies only in case the consumer is an individual. Laica products

are designed for domestic use only; therefore, their use for commercial purposes is forbidden.

The warranty covers only production defects and does not apply, if the damage is caused by accidental events, misuse, negligence or improper use of the product. Use the supplied accessories only; the use of other accessories invalidate the warranty cover.

Do not open the equipment for any reason whatsoever; the opening or the tampering of the equipment, definitely invalidate the warranty.

Warranty does not apply to parts subject to tear and wear along the use, as well as to batteries, when they are included. Warranty decays after 2 years from the purchase date. In this case, technical service interventions will be carried out against payment. Information on technical service interventions, under warranty terms or against payment, may be requested by writing at info@laica.com. Repair or replacement of products covered by the warranty terms are free of charge. In case of faults, refer to the dealer; DO NOT send anything directly to LAICA.

All interventions covered by warranty (including product replacement or the replacement of any of its parts) shall not be meant to extend the duration of the original warranty period of the replaced product. The manufacturer cannot be made liable for any damages that may be caused, directly or indirectly, to people, things and pets resulting from the inobservance of the provisions contained in the suitable user manual, especially with reference to the instructions related to installation, use and maintenance of the equipment.

Being constantly involved in improving its own products, Laica reserves the right to modify its product, totally or partially, with reference to any production need, without that this involves any responsibility whatsoever by Laica or its dealers.

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Guidance and declaration of manufacturer-electromagnetic emissions

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.

Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The device uses RF energy only for its internal function. Therefore, its emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	N/A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	N/A	

Guidance and manufacturer's declaration – electromagnetic immunity

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines	± 2 kV for power supply lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV line(s) to line(s)	± 1 kV line(s) to line(s)	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% UT (>95% dip in UT) for 0.5 cycle 40% UT (60% dip in UT) for 5 cycles 70% UT (30% dip in UT) for 25 cycles <5% UT (>95% dip in UT) for 5 s	<5% UT (>95% dip in UT) for 0.5 cycle 40% UT (60% dip in UT) for 5 cycles 70% UT (30% dip in UT) for 25 cycles <5% UT (>95% dip in UT) for 5 s	Mains power quality should be that of a typical commercial or hospital environment. If the user of the device requires continued operation during power mains interruptions, it is recommended that the device be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

Guidance and manufacturer's declaration - electromagnetic immunity

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 V	<p>Portable and mobile RF communications equipment should be used no closer to any part of the device, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance: $d = 1,2\sqrt{P}$ 80 MHz to 800 MHz $d = 2,3\sqrt{P}$ 800 MHz to 2,5 GHz</p> <p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, a should be less than the compliance level in each frequency range b.</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol: </p>
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2,5 GHz	3 V/m	

Recommended separation distances between portable and mobile RF communications equipment and the device

The device is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the device as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1,2\sqrt{P}$	80 MHz to 800 MHz $d = 1,2\sqrt{P}$	800 MHz to 2,5 GHz $d = 2,3\sqrt{P}$
0,01	0,12	0,12	0,23
0,1	0,38	0,38	0,73
1	1,2	1,2	2,3
10	3,8	3,8	7,3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be determined using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.
 NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.
 NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.